
ConC 400

EN Welcome  
 DE Willkommen  
 FR Bienvenue  
 ES Bienvenidos  
 PT Bem-vindo  
 NL Welkom  
 IT Benvenuto  
 DA Velkommen  
 SV Välkommen


Quick Guide | Kurzanleitung | Guide rapide | Instrucciones resumidas | Guia rápido | Beknopte handleiding | Guida rapida | Kort vejledning | Snabbguide

Model: ConCPlus1, ConCPlus1-R, ConCPlus1-L, ConCPlus1-C

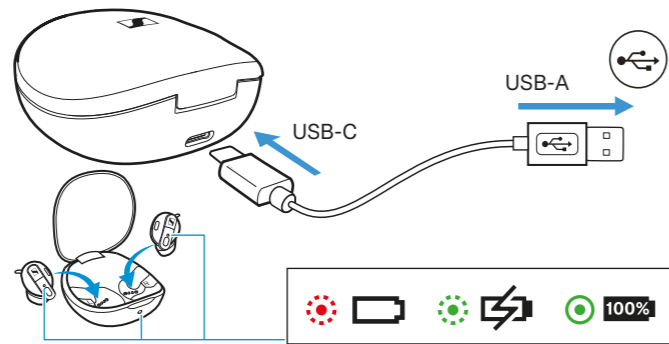
**1. Waking up and charging the rechargeable batteries** | Akkus aufwecken und laden | Réveiller et charger les batteries | Despertar y cargar las pilas recargables | Ativar e carregar as pilhas recarregáveis | De accu's plaatsen en opladen | Attivazione e caricamento delle batterie ricaricabili | Vækning og opladning af batterier | Väcka och ladda batterierna


**2. Starting the Bluetooth® pairing** | Bluetooth®-Kopplung starten | Démarrer l'appairage Bluetooth® | Iniciar la sincronización Bluetooth® | Iniciar o emparelhamento Bluetooth® | Bluetooth®-koppeling starten | Avvio dell'accoppiamento Bluetooth® | Start af Bluetooth®-kobling | Starta Bluetooth®-parkoppling

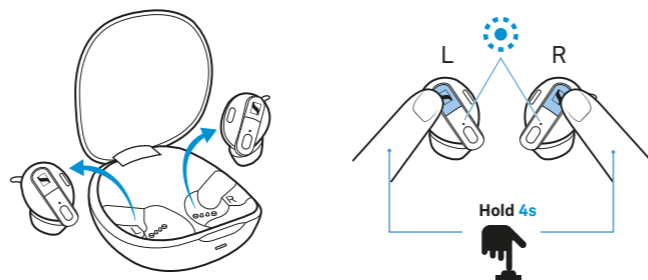
**3. Establishing a connection** | Verbindung herstellen | Établir une connexion | Establecer la conexión | Estabelecer a ligação | De verbinding tot stand brengen | Creazione di una connessione | Etablering af forbindelse | Upprätta en anslutning

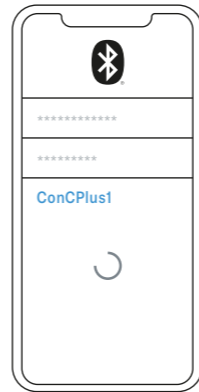**4. Selecting ear tips and ear fins and inserting the earbuds into the ears** | Ohradapter und Ohrfinnen wählen und Hören in die Ohren einsetzen | Sélectionner les embouts d'oreille et les ailettes d'oreille et insérer les écouteurs dans les oreilles | Seleccionar los adaptadores para los oídos y las aletas y colocar los audífonos en los oídos | Escolher os adaptadores e alhetas auriculares e colocar os auscultadores nos ouvidos | Ooradapters en oorvinnen kiezen en oortelefoons in de oren inbrengen | Scelta dei gommini e delle alette auricolari e inserimento delle microcuffie nelle orecchie | Valg af øreadaptere og ørefinner og isætning af øretelefoner i ørerne | Välja öronadaptrar och öronfenor och sätta in hörlurarna i öronen

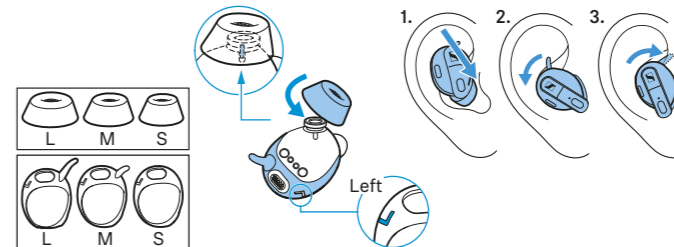**5. Downloading the app for customized settings of the earbuds** | App herunterladen für individuelle Einstellungen der Ohrhörer | Télécharger l'appli pour les réglages personnalisés des écouteurs | Descargar la app para los ajustes individuales de los audífonos | Descarregar a app para definições individuais dos auriculares | App downloaden voor individuele instellingen van de oortelefoon | Download dell'app per impostazioni individuali delle microcuffie | Download af app til individuelle indstillinger af øretelefonerne | Ladda ner app för individuella inställningar av hörlurarna

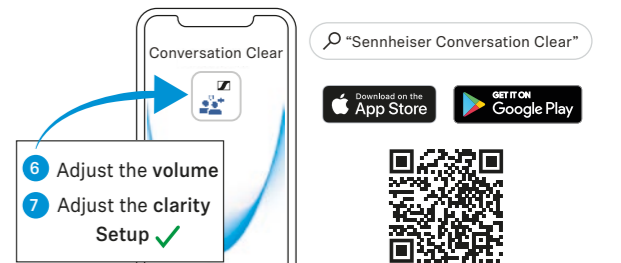www.sennheiser-hearing.com/conversation-clear-app

Instruction manuals and product support | Bedienungsanleitungen und Produktbetreuung | Notices d'emploi et support produit | Instrucciones de manejo y asistencia al producto | Manuais de instruções e assistência ao produto | Gebruiksaanwijzingen en product-support | Istruzioni per l'uso e assistenza sul prodotto | Betjeningsvejledninger og produktrådgivning | Bruksanvisningar och produktguide

[www.sennheiser-hearing.com/conc-400](http://www.sennheiser-hearing.com/conc-400)

Sonova Consumer Hearing GmbH  
 Am Labor 1, 30900 Wedemark, Germany  
[www.sennheiser-hearing.com](http://www.sennheiser-hearing.com)

029-3353-72/V1.03/2022-12

Touch controls | Touch-Bedienflächen | Panneaux de commande tactile | Paneles de manejo táctiles | Painéis de controlo táteis | Touch-betjeningsvlakken | Pulsanti touch | Touch-betjeningsflader | Manövertytor

	Relax	Communication	Stream
Press / Hold		Left	Right
Short press 1x		Volume -	Volume +
Hold 2s		—	Communication Mode
Tap 2x		Voice assistant	Play

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	Relax	Communication	Stream
		Left	Right
Short press 1x		Volume -	Volume +
Hold 2s		Relax Mode	Clarity boost on/off
Tap 2x		Voice assistant	Play

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	Relax	Communication	Stream
		Left	Right
Short press 1x		Ambient balancing	Ambient balancing
Hold 2s		Relax Mode	Communication Mode
Tap 2x		Voice assistant	Play / Pause

Controlo dependendo do modo de utilização: Relaxar („Relax“, apenas ANC ativo), Comunicar („Communication“, contacto com o ambiente, sem reprodução de música), Transmitir („Stream“, reprodução de música).  
 Bediening afhankelijk van de gebruiksmoedus: Ontspannen („Relax“, alleen ANC actief), Communiseren („Communication“, contact met de omgeving, zonder afspelen van muziek), Streamen („Stream“, muziek afspelen).

	Phone calls
	Left or Right
Short press 1x	Accept call
Tap 2x	Reject call
Hold 2s	End call
Tap 2x	Ambient balancing

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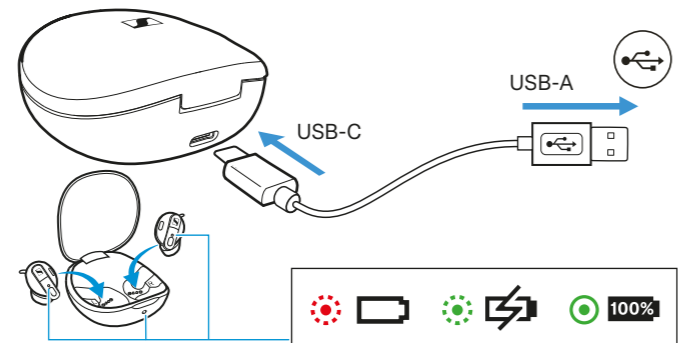
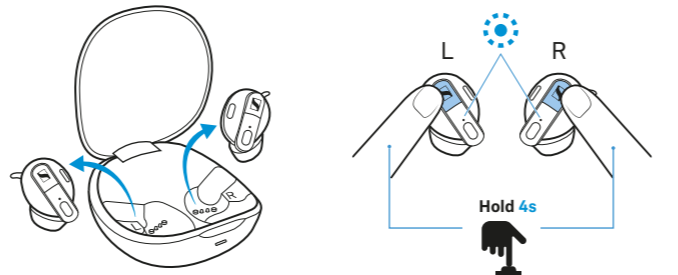
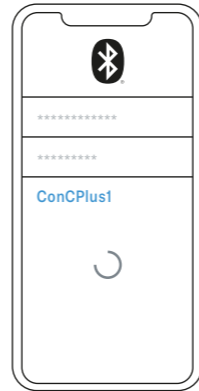
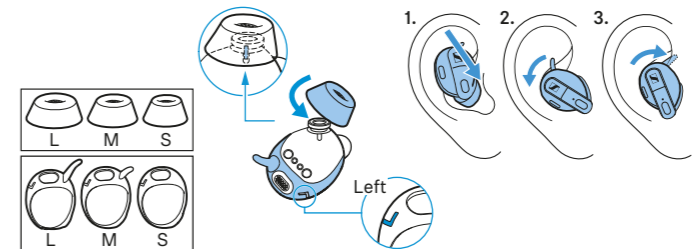
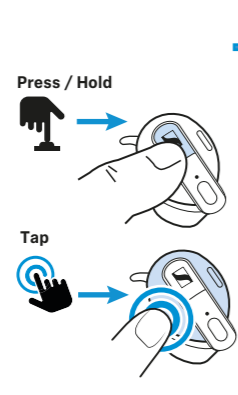
	Relax	Communication	Stream
Press / Hold		Left Right	
Short press 1x		Volume -	Volume +
Hold 2s		—	Communication Mode
Tap 2x		Voice assistant	Play

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	Relax	Communication	Stream
		Left Right	
Short press 1x		Volume -	Volume +
Hold 2s		Relax Mode	Clarity boost on/off
Tap 2x		Voice assistant	Play

„Sterowanie w zależności od trybu użytkowania: Relaks („Relax“, aktywne tylko ANC), Komunikacja („Communication“, kontakt ze środowiskiem, bez odtwarzania muzyki), Streaming („Stream“, odtwarzanie muzyki).“  
 Kullanım moduna göre kontrol: Rahatlama („Relax“, yalnızca ANC etkin), iletişim („Communication“, müzik çalma işlemi olmadan ortam ile iletişim), Akış („Stream“, müzik çalma).


	Relax	Communication	Stream
		Left Right	
Short press 1x		Ambient balancing	Ambient balancing
Hold 2s		Relax Mode	Communication Mode
Tap 2x		Voice assistant	Play / Pause

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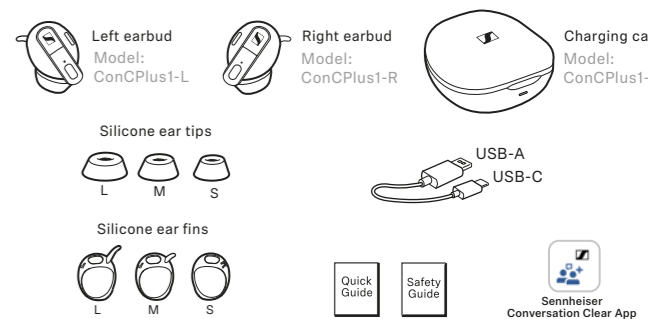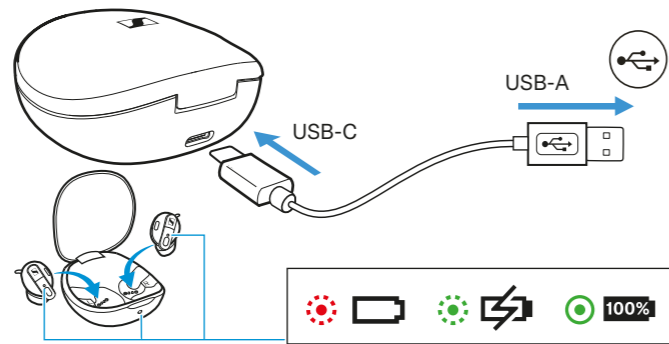
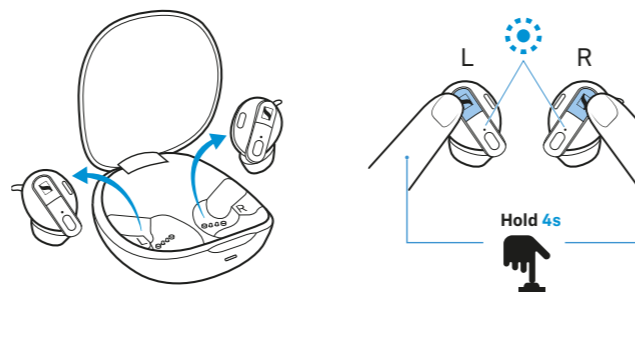
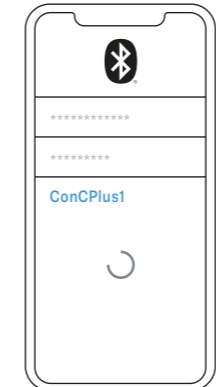
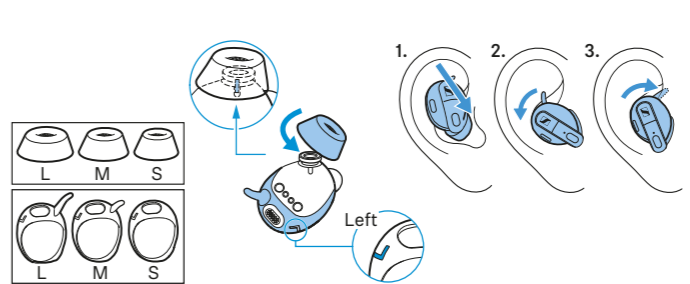
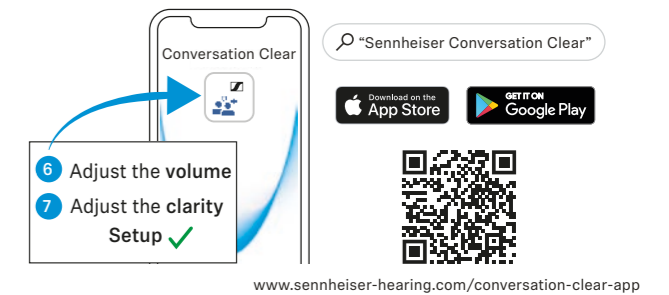
	Relax	Communication	Stream
Press / Hold	Short press 1x	Volume -	Volume +
Tap	Hold 2s	—	Communication Mode
	Tap 2x	Voice assistant	Play

	Relax	Communication	Stream
Press / Hold	Short press 1x	Volume -	Volume +
Tap	Hold 2s	Relax Mode	Clarity boost on/off
	Tap 2x	Voice assistant	Play

	Relax	Communication	Stream
Press / Hold	Short press 1x	Ambient balancing	Ambient balancing
Tap	Hold 2s	Relax Mode	Communication Mode
	Tap 2x	Voice assistant	Play / Pause

	Phone calls
Press / Hold	Accept call
Tap	Reject call
Press / Hold	End call
Tap	Ambient balancing

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